Prior to use, carefully read the instructions for use.

EN

GC Fuji COAT LC

LIGHT CURED PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- a) Place the glass ionomer cement according to the manufacturer's instructions.
- b) Using a cotton pellet or brush apply GC Fuji COAT LC to all exposed glass ionomer and adjacent surfaces.
- c) Light cure for 10 seconds using suitable visible light curing device (>500mW/cm²).
- d) Close bottle immediately after use.

STORAGE

Store in a cool and dark place (4-25°C) (39,2-77,0°F). (Shelf life : 2 years from date of manufacture).

PACKAGE

Bottle of 5g (5.2mL)

CAUTION

1. GC Fuji COAT LC is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
2. In case of contact with oral tissue or skin, flush immediately with water.
3. In case of contact with eyes, flush immediately with water and seek medical attention.

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